ECSA MAINTENANCE AG

Via Luigi Favre 16 | CH-6828 Balerna (TI) - Switzerland
 Burgauerstrasse 17 | CH-9230 Flawil (SG) - Switzerland
 phone: +41 58 211 93 00

COPRISCARPE IN TNT MONOUSO NON STERILE NICHT STERILE EINWEG-SCHUHABDECKUNG IN TNT NON STERILE DISPOSABLE TNT SHOE COVER

HERSTELLER: ECSA MAINTENANCE AG Via Luigi Favre 16 CH-6828 Balerna (TI) Switzerland

Artikel:
AI 006398

COPRISCARPE MONOUSO
EINWEG-SCHUHABDECKUNG
DISPOSABLE SHOE COVER

MODELL-NR.:

PF10015

Batch-Nr.:

LOT

0002

Herstellungs-
datum:

30/10/2020

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ENGLISH

USER MANUAL Version: 0 - Last version date 24/08/2020**DISPOSABLE TNT SHOE COVER Cod. PF10015****DESCRIPTION OF THE PROTECTIVE DEVICE**

The disposable non-woven shoe cover is a mechanical barrier against materials and substances, it serves to prevent contamination from minor agents and offers protection from dust and dirt. It is equipped with a latex-free elastic seam and is not sterile.

Field of application and intended use.

The device offers protection from dust and dirt. The combination with visors, gowns, gloves and disposable caps is recommended.

Usage

- Sanitize your hands by washing them abundantly with soap and water or with a hydroalcoholic disinfectant solution.
- Remove the overshoe from the package.
- Insert the shoe cover by spreading the elastic so as to completely cover the shoe.
- To remove the shoe cover, carry out the operations described in reverse.

ATTENTION:

- ☐ When worn correctly, the shoe cover should be comfortable and comfortable, without impeding or restricting any movement.
- ☐ The shoe cover can be worn continuously for up to 20 hours, after which it must be taken off to allow the user's skin to breathe.

Disposal

After using the device, proceed with its disposal.

The construction materials of the devices do not require special disposal procedures. It is necessary to refer to the local regulations for the disposal of unsorted waste. In case of use in hospitals, refer to the legislation for the disposal of special waste with risk of contamination. Do not dispose or abandon the product in the environment for any reason.

TECHNICAL SPECIFICATIONS**Materials:**

- Fabric: 100% polypropylene 30 gr / mq SPUNBONDED non-woven fabric
- Elastic band: 50% POLYESTER, 20% NYLON, 30% NATURAL RUBBER

SIZE AND FIT

The overshoe is available in standard size with an elastic band.

Base length 37 cm

Base width 25 cm

Total height 18 cm (tolerances $\pm 2\%$ for all dimensions)

USE**Storage:**

- Store the hoods in a cool indoor environment, away from bad weather and jets of steam.
- Store the shoe covers away from heat sources, open flames and direct sunlight.

DISPOSAL

After using the shoe cover, proceed with its disposal.